



## Instadose – Web browser settings

**Sid 1) Internet Explorer**

**Sid 2) Google Chrome**

**Sid 3) Mozilla Firefox**



## Internet Explorer

- 1). Log in to [www.instadose.com](http://www.instadose.com). Download and install the software which is required to read the Instadose dosimeter. This requires you have **administrator** rights on your computer.
- 2). Create a **company** and **wearers** adding the mandatory personal information required by Instadose. If the wearer is a category A worker, additional information is required to report to NDR (Svedos). Assign a dosimeter to each wearer. It's not necessary a wearer is a person, it could a visitor or background dosimeter as well.
- 3). Activate dosimeter with **Initialize Device**

Read Device

## Web browser settings

When Instadose is read, the result is shown as a picture. If a picture is not shown, please try:

*Please note:* the result is always registered if the computer has Internet and the dosimeter is assigned.


### Manage

[Account Profile](#): Update the pertinent information on your account.  
[Devices](#): View devices in your account and assign them to wearers.  
[Users](#): Add and modify wearers.  
[Locations](#): Add and modify locations.  
[Download Software](#): Download instadose reader software to perform reads from your computer.  
[Registration Wizard](#): Set up your instadose account and devices through this wizard.  
[Initialize Device](#): Initialize your device before wearing and performing reads.


### Reports

[Reading History](#): View the most recent reads in your account.  
[History Detail](#): Get a full history of readings for a wearer.  
[Radiation Exposure](#): View the radiation exposure report.  
[Dosimeters Not Read](#): View users that have not read their dosimeters.

- A. Allow Pop-Up from the adress \*.instadose.com

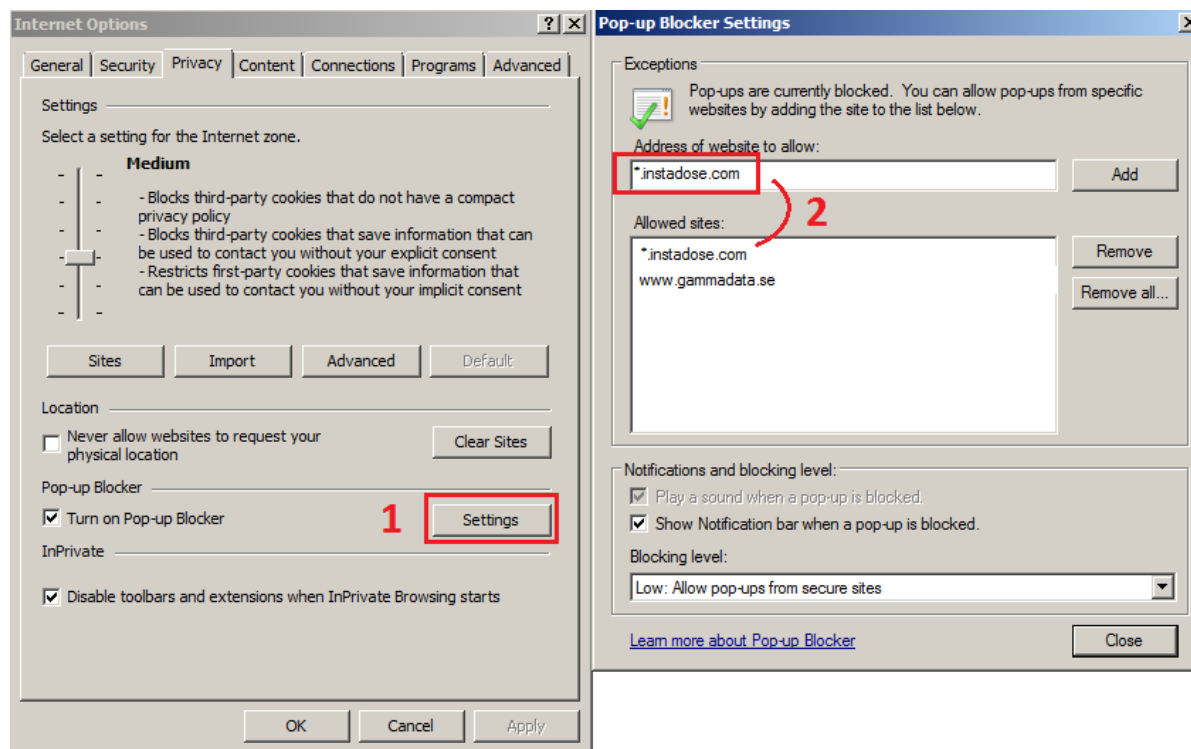

B. If it still does not work, allow cookies at [www.instadose.com](http://www.instadose.com)

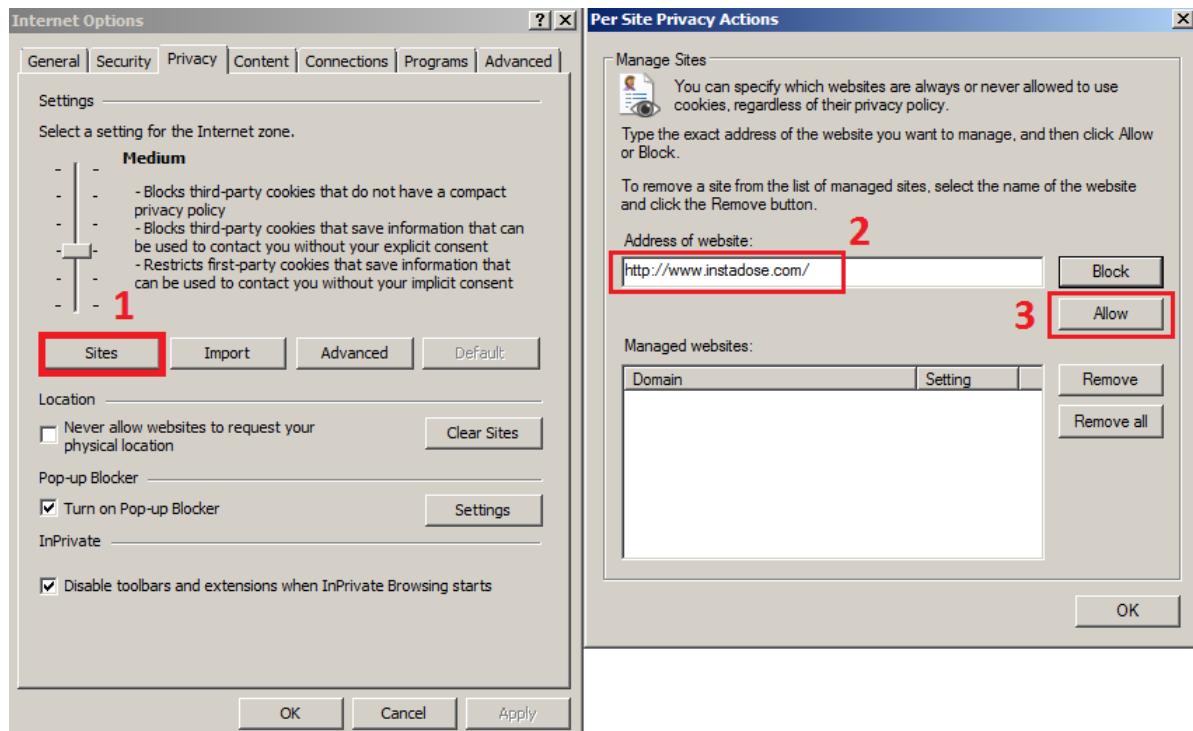

## Google Chrome

- 1) Log in with your account at [www.instadose.com](http://www.instadose.com). Download and install the software which is required to read the Instadose dosimeter. This requires you have **administrator** rights on your computer.
- 2) Create a **company** and **wearers** adding the mandatory personal information required by Instadose. If the wearer is a category A worker, additional information is required to report to NDR (Svedos). Assign a dosimeter to each wearer. It's not necessary a wearer is a person, it could a visitor or background dosimeter as well.
- 3) Activate dosimeter with **Initialize Device**

## Web browser setting Chrome (from ver. 90)

When Instadose is read, the result is shown as a picture. If a picture is not shown, please try:

*Please note:* the result is always registered if the computer has internet and the dosimeter is assigned. See your result in *Reading history*

### Manage

[Account Profile](#): Update the pertinent information on your account.

[Devices](#): View devices in your account and assign them to wearers.

[Users](#): Add and modify wearers.

[Locations](#): Add and modify locations.

[Download Software](#): Download instadose reader software to perform reads from your computer.

[Registration Wizard](#): Set up your instadose account and devices through this wizard.

[Initialize Device](#): Initialize your device before wearing and performing reads.

### Reports


[Reading History](#): View the most recent reads in your account.

[History Detail](#): Get a full history of readings for a wearer.

[Radiation Exposure](#): View the radiation exposure report.

[Dosimeters Not Read](#): View users that have not read their dosimeters.

- 1) Allow Pop-up and cookies at \*.instadose.com by
  - a. Go to Settings (three dots I upper right corner),
  - b. go to *Privacy and Security*,
  - c. *go to Cookies and other site data*,
  - d. Add \*.instadose.com to *Sites that can always use cookies*

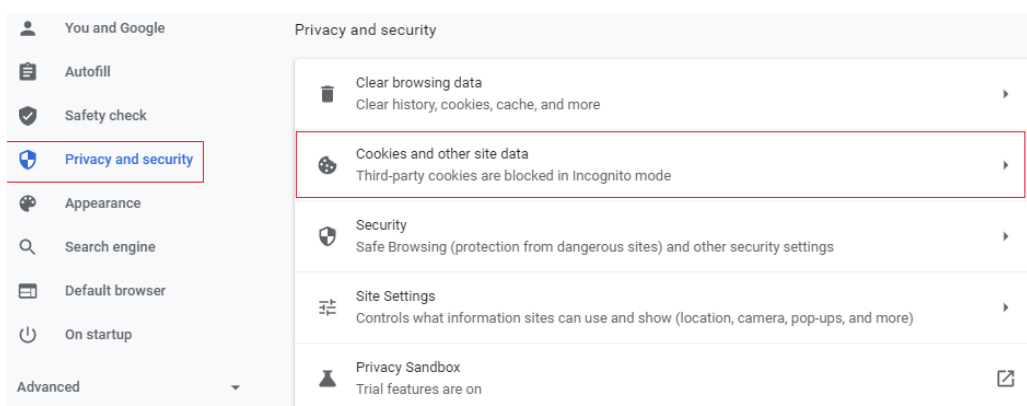

Sites that can always use cookies

Add

 [www.instadose.com](http://www.instadose.com)  
Including third-party cookies on this site



## Mozilla Firefox

No known problems when using Mozilla Firefox